
Suggested Electrode Placement Chart

The sticky electrode pads need to be placed in the correct position for maximum pain relief. It is preferable to obtain assistance from either your birthing partner or medical adviser when positioning the pads. Please read the following instructions carefully:

- 1. Ensure unit remains switched off. Carefully peel the pads from the plastic backing using your forefinger and thumb. Do not pull on the lead wires.
- 2. Position the top two pads (connected to the left socket CH1) approximately 4cm either side of the spine and 4-5cm above the imaginary navel line as seen in the diagram.
- 3. Position the bottom two pads (connected to the right socket CH2) approximately 4cm either side of the spine and 4-5cm below the imaginary navel line as seen in the diagram.

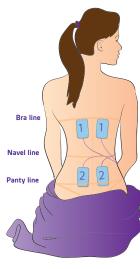

Electrodes can be used several times but the number of uses varies according to the condition of the skin

Reference: Cluett E. (1994) SRN RM ADM PGCEA. Analgesia in Labour: A review of the TENS method. Professional Care of Mother and Child. Mar: 4 (7): 50-2

Using Your Elle TENS 2

a) You are now ready to start using your Elle TENS 2. Switch the unit on by pressing the @on/off button. Ensure that the 'boost' button located at the top right of the unit is NOT pressed in. The Elle TENS 2 will always start up in BURST 1 mode.

b) Begin by pressing the CCH1+ key. Each time you press this key, a small 'half-moon' shape will appear on the screen increasing in number and size as the intensity is increased. You will begin to feel a pulsating sensation which will burst in time with the small arrow flashing on the screen. Ensure that you increase the intensity slowly.

c) Repeat the previous step with the CH2+ key. You will now feel a sensation in both sets of pads.

d) BURST 1 (use between contractions): This is the first mode that you will use. It feels like a bursting, tingling sensation i.e. the pulses are off and on periodically. This mode will help to promote those all-important natural pain-killing chemicals known as 'endorphins' and 'encephalins'.

Recalling your contraction history

To recall a list of your last 5
'Contraction Times' (CT) and
Between Contraction Times' (BCT)
press the God CHI- and CH2buttons simultaneously. The last
recorded set of BCT and CT times
are shown as 1

Use CH2+ to go back to previous sets (eg, 2, 3, 4, 5) or forward again by pressing CH2-key. Recalling the contraction history will not interrupt the functionality of the unit. N.B. To avoid losing contraction history, do NOT switch unit off. To return back to the original mode screen, (which will have continued counting in the background) press the CH1- and CH2- buttons together again.

The Mode Button

There are two mode ranges available on the Elle TENS 2, each with a Burst and Boost function.

Mode 1 (BURST 1 and BOOST 1), which has been described above, is primarily used for the earlier stages of labour.

Mode 2 (BURST 2 and BOOST 2) usually used when your labour

advances and your contractions become more frequent.

Press the Mode Button to switch between mode ranges. You will see from the User Display, that the number on the centre right of the display will change from 1 to 2 (and vice-versa). N.B. Both modes are interchangeable and can be used according to your preference, not necessarily for the stage of labour you are in.

The Opti-Max Key

This provides extra versatility to your Elle TENS 2. Increasing or decreasing the Opti-Max Key will allow you to either heighten or lessen the level of pulse power transmitted to the electrode pads. Pressing the Opti-Max Key either + or - will increase or decrease Opti-Max sensation. This is extremely useful for the final stages of labour when you need an extra surge of power. There are 5 Opti-Max levels: 50, 100, 150, 200 and $250\mu s$ (microseconds). The unit will always start up at Opti-Max 150 μ s, which is exactly half way between the highest and lowest setting. N.B. The Elle TENS 2 starts up in BURST 1. The BCT only appears after one cycle of pressing the Boost button on and off.

S S N 3 T 3 1 1 3

Succision for use f

Babycare TENS

7

Frequently Asked Questions

Q When shall I start using my Elle TENS 2?

A Use as early as you can after the onset of labour to allow time for your body's painkilling chemicals to rise.

Q Can I try the Elle TENS 2 before going into labour?

A Yes, place 2 pads attached to one leadwire on your forearm to feel the different pulse sensations following the instructions on page 8.

Q Can I combine the Elle TENS 2 with other medication?

A Yes, you can still use analgesics such as gas and air (entinox) or pethidine.

Q Can the Elle TENS 2 be used in hospital?

A Yes. However, a midwife may ask you to switch it off temporarily.

Q Can I use the Elle TENS 2 if I am considering a water birth?

A Yes, but not in the water itself.

Troubleshooting Questions

Q Why does the stimulation not appear to be as strong after you have used the Elle TENS 2 for a while?

A Increase the intensity, you may have become used to a lower setting.

Q Why does the stimulation feel strong but ineffective?

A You may need to switch off the unit and reposition the electrode pads.

Q Why does the stimulation not feel sufficiently strong even on a high setting?

A The batteries may need replacing.

Q What do I do if there is no stimulation, intermittent stimulation, or weak stimulation felt even wit a new battery?

A Check electrode contact. Check leads. Check batteries. Call our customer careline on +44 (0)20 8532 9595.

Useful Information

 Continue stimulation for as long as necessary, unless you experience discomfort.

If you stimulate for a number of hours ensure the pads do not become dry. If this occurs, switch off the unit, disconnect the electrodes, and apply a small drop of water to them.

3. The pads can be used several times. Replace the pads when they stop sticking to the skin. Replacement pads are available from www.babvcaretens.com.

 Always ensure that the unit is switched off before you remove all the leads and pads. Do not pull the wires. ALWAYS remove the jack plug from its socket by holding the plug between the forefinger and thumb and pulling gently.

5. The ElleTENS 2 can also be used as a standard TENS unit for relief of numerous conditions, including after pains and post operative pains It is worth bearing this in mind if you have a caesarean section. Please seek medical advice first.

Post-Natal Pad Placement Diagrams

KEY:

=first pair of electrodes

e) BOOST 1 (use during

contractions): When experiencing a

contraction, simply press the 'boost'

button located at the top right of the

unit. The Elle TENS 2 will then switch

automatically into the BOOST 1

contractions. This will feel like a

continuous sensation.

mode for that extra surge of power

required to combat the pain during

f) Once the contraction has passed,

simply release the 'boost' button. The

Elle TENS 2 will return to the original

BURST 1 mode. Continue this cycle

during your early stages of labour.

Using the Contraction Timer

The first time you press the Boost

automatically trigger the Contraction

Timer (CT). This can be seen in the

bottom right of the display. This

contractions in seconds. Once the

contraction has passed and the Boost

measures the duration of your

Button is released, the unit will

trigger the Between Contraction

Timer (BCT) which measures the


he seen in the hottom left of the


time between contractions. This can


Button, the Elle TENS 2 will

Second pair of electrodes.

N.B. Electrodes supplied with this unit are all the same colour - the black and white squares show suggested electrode placement only.

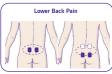

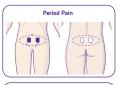

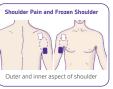

Elle TENS 2 Technical Data

	Programme	Display	Output Frequency / Duration
ENS lant with EEC Directive 93/42/EEC, powered equipment type BF.	Burst	Burst 1	Pulse Frequency (16Hz) Pulse Width (150 μs) 1 burst per second, 8 pulses per burst*
nuous operation. dangad (pulse width not danged when switched between BURST les widths in at modes: 50, 1 00, 1 50, 200, 2 50	Burst	Burst 2	Rise Frequency (33Hz) Rise Width (150 µs) 2 bursts per second. 8 pulses per burst*
115mm x 55mm x 30mm	Boost	Boost 1	Pulse Frequency (80Hz) Pulse Wildth (150 µs) continuous*
75 grams			Duling Exercises (2001) Duling Middle (2001)
CE Touch-proof Jacks	Boost 2	Boost 2	ruse rrequeity (100rg) ruse widti (150 gs)
Digital LCD - 8 seven segment digits and around 95 segments	Output Channel	_	Dual Channel
Will switch off after 5 minutes if not used			
2.2v±0.2v	Output Waveform	E	Symmetrical bi-phasic rectangular
2 x 15v AA Battery (Type LR6)	OutputVoltage		0 - 55v zero to peak adjustable in 15 steps at 500 Ω los
Temperature 0.40°C Air humldity 30-75% Air pressure 700 hPa-1060 hPa	Output Intensity		0 -110mA adjustable in 15 steps at 500 Ω load intensitevel will drop back to zero after programms changes
Temperature 0-40°C Air humidity 30-90% Air pressure 700 HPa-1060 HPa	Pulse width adjustment of Default pulse width is 150	ment does r is 150 µrs. T	Pulse width adjustment does not change when switching between Burst and Boost mode Medatir palve with 1550 gas. There are 5 adjustable pulse withis selectable in all modes; from one or 5 adjustable pulse withis selectable in all modes;
	100, 100, 200 gilds	9	

Guarantee Your Elle TENS 2 machine is guaranteed for a period of 1 year against manufacturer's defects. Please record purchase details in the space provided below for your record purposes. The guarantee does not include leads, electrodes or battery. Applies to purchases only. Date of purchase: Invoice number:

Babycare TENS°

Body Clock Health Care Ltd

108 George Lane, South Woodford, London E18 1AD

Tel (+44) (0)20 8532 9595 Fax (+44) (0)20 8532 9551 www.babycaretens.com e-mail customercare@babycaretens.com

EC REP MedEnvoy Global B.V.
Prinses Margrietplantsoen 33,

Prinses Margrietplantsoen 33, Suite 123, 2595 AM The Hague, The Netherlands

Babycare TENS is part of the Body Clock group of companies Oct 2023 (6)
Model: D-BC138T
ELLE2- IFU-1

recycle

1639

This manual is recyclable

10

FILE TENS 2 Quick Reference Guide

If you are in labour, please use the quick reference quide set out below. It is however recommended that you read the section entitled "Setting Up Your Elle TENS 2" found on page 5 to obtain the best performance from vour unit.

Before using the Elle TENS 2, please read the contraindications and warnings on pages 1 and 2.

- 1. Ensure the Elle TENS 2 unit is switched off
- 2. Insert 2 x AA batteries in the compartment at the back of the unit (see figs 1 and 2, page 6).
- 3. Insert the leadwire plugs into the top of the unit (see fig 3, page 6).
- 4. Insert the pins at the other end of the leadwires into the sockets attached to the electrode pads. Do not remove the electrodes from their backing at this stage (see fig 4, page 6).

Controls For The Elle TENS 2

LCD lit by

automatio

back light

Opti-Max Kev

Adjusts depth of

Adjustment Kevs

Mode Button

Selects modes

neck cord

Space to attach

the pulses

Channel 1

Intensity

instructions to obtain the most out of the unit.

been read thoroughly. See pages 1 and 2.

The Elle TENS 2 is very simple to use. However, it is essential to read these

Please ensure that all of the contraindications and precautions have

FLLETFNS2

▶BURST 3 ■

▶BOOST ∂

BBmBBs Spentill 150m

Babycare TENS

0

- 5. Position the electrode pads on the body as indicated on page 7. A pair of electrodes must be connected to each lead wire in order for you to feel any sensation. The electrode pads must be peeled away from their backing and placed directly on your body for you to feel any sensation
- 6. Make sure that the 'boost' button at the side of the unit is NOT pressed in. This ensures that the unit will start in the BURST 1 mode (see page 3).
- 7. Press the (1) (on/off) to turn the unit on
- 8. Now push the (CH1 +) key until you feel a pulsating sensation
- 9. Repeat the same with the (CH2+) key.
- 10. At the onset of a contraction press the 🗲 'boost' button for the additional surge of power needed to combat the pain.
- 11. Release the 'boost' button once the contraction has passed and continue in the BURST 1 mode

Leadwire Jack

Plug Sockets

Boost/Timer

Used for an

extra surge of

power during

contractions/

On/Off Buttor

Channel 2

Adjustment

Intensity

Kevs

When the intensity

FILE TENS 2 has not

been in use for 5

automatically

levels are zero, and the

minutes, it will shut of

N R Automatic

Power Off

Button

timing

Contents

Elle TENS 2 Quick Reference Guide How Will The Elle TENS2 Help Your Pain?..1 Introduction To The Elle TENS 2 General Precautions ... Controls For The File TFNS 2 Display Symbols and their Meanings 4 Setting Up Your Elle TENS 2......5 Useful Diagrams..... Suggested Electrode Placement Chart...... 7 Using Your Elle TENS 2..... Frequently Asked Questions 10 Troubleshoooting Ouestions 10 Useful Information Post natal pad placement diagrams 11 Technical Specifications

Explanation of Symbols on Unit



Equipment providing a particular degree of protection against electric shock particularly regarding allowable leakage currents having an F type (floating) applied part.



Warning - refer to pages 1-2 of these instructions.



EC REP EU Representative.



This unit needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENT.

SN

Display Symbols And Their Meanings

▶BURST

BURST is used at the onset of labour and in between contractions There are two Burst modes available indicated by a 1 or 2. You can switch between them by pressing the Mode Button (see page 9).

▶BOOST

The Boost Button at the top right of the unit activates the BOOST mode. This function is used during contractions. There are two Boost modes available indicated by a 1 or 2. You can switch between them by pressing the Mode Button



BETWEEN CONT RACTION TIMER Starts by pressing the Boost Button, this automatically measures the time interval between your contractions

▶BURST / ▶BURST 2

MODE 1 AND 2 There are two mode ranges available each with a Burst and Boost function (see page 9).



LOW BATTERY INDICATOR This sign will start flashing if the hatteries run low. If this is the case change both batteries. Please refer to page 5 and 6 for instructions.

Please keep device away

IP22 The first number 2:

from sprays of water or rain.

Protected against access to

hazardous parts with a finger.

and the jointed test finger of

12 mmΦ, 80 mm length, shall

have adequate clearance from

hazardous parts, and protected

against solid foreign objects

Protected against vertically

falling water drops when

enclosure tilted up to 15°

Vertically falling drops shall

have no harmful effects when

the enclosure is tilted at any

angle up to 15° on either side of

Denotes a product which must

be disposed of safely.

This symbol indicates the

compartment and outer

serial number of the device.

It can be found in the battery

packaging. It includes the year

of manufacture (YOM) which

appears as two digits after the

letters FD e.g. FDXX. The digits

following the YOM identify

each individual device.

second number 2:

the vertical.

of 12.5 mm Φ and greater. The



POWER BAR

This shows the output from 0 to 15 for both channel 1 and 2. Each segment represents approx 7% of the total output.



CONTRACTION TIMER Starts by pressing the Boost Button, this automatically records the duration of your contractions.

Opti Max util 250 us

OPTI-MAX Indicates the Opti-Max output. Refer to page 9 for more details.



PAD CONTACT WARNING (safety cut-out) This symbol will appear if any of your pads are not attached securely to the skin

How Will The Elle TENS 2 Help Your Pain?

TENS (Transcutaneous Electrical Nerve Stimulation) works by passing mild electrical impulses through the skin, via electrode pads, into the nerve fibres which lie below. The TENS impulses help your body produce its own pain killing chemicals, such as 'endorphins' and 'encephalins'

The level of pain relief obtained varies from person to person. Some find that the Elle TENS 2 provides all the pain relief required during the whole of labour, whereas others find that other analgesics are also needed during the later stages of

Introduction To The Elle TENS 2

The Elle TENS 2™ is an efficient method of drug-free pain relief. This powerful digital TENS unit is smooth, sophisticated and simple to use. It has been cleverly crafted by Babycare TENS specifically for home use for mums-to-be, for the management of pain for labour and beyond.

Precautions And contraindications

- 1. TENS must NOT be used before the 37th week of pregnancy.
- 2. If you are pregnant, do NOT place the electrodes over your
- Do NOT use if you have a pacemaker or any other implanted electrical device.
- Check with your healthcare professional before using if you suffer from a heart condition, epilepsy, undiagnosed pain, have any metal implants, or any doubts whatsoever.
- Do NOT use if you experience an allergic reaction to the electrodes
- Never use TENS to mask undiagnosed pain since this could require urgent treatment

General Precautions

- 1. Do NOT use this unit without first reading these instructions.
- 2. Do NOT immerse the Elle TENS 2 in any liquid. Do not use in the bath or the shower.
- 3. Do NOT place it close to any source of excessive heat or operate it in the presence of flammable gas.
- 4. Do NOT drop this unit onto a hard surface.
- 5. Do NOT attempt to dismantle the Elle TENS 2.
- 6. Only use specified batteries and electrodes.
- 7. If damaged, do not use. Return to supplier.
- Remove the batteries when not in use
- 9. Do NOT use while driving or operating potentially dangerous machinery or while using a microwave.
- 10. Keep out of the reach of children.

- 11. Do NOT place electrodes on or near the eves, in the mouth, over the front or sides of the neck or across the head or the
- 12. Do NOT place the electrodes on varicose veins or recent scarring or on any area of broken, inflamed, infected or numb skin. Electrodes should generally only be applied to skin with normal sensation unless under medical supervision.
- 13. Do NOT use in the presence of tuberculosis, malignant tumours, very high or very low blood pressure, high fever or acute inflammatory disease unless under medical supervision.
- 14. A mild shock may be caused if one electrode is accidentally removed. In the event of this happening the unit should be turned off immediately

NB. The unit should be wiped clean periodically using medically approved anti-bacterial wipes or a very slightly dampened cloth. A mild soap may be used but DO NOT apply solvents.

Sticky Electrodes

Setting Up Your Elle TENS 2 Check the following contents:

- 1 x Elle TENS 2 Unit
- 1 x Pack of 4 Self Adhesive Electrodes (40mm x 100mm)
- · 2 x Leadwires
- 2 x AA Batteries
- · 1 x Carrying Pouch
- 1 x Easy-Release Neck Cord
- 1 x Instruction Manual
- 1 x EMC documentation

The Assembly Stage

Make sure the unit is swiched off.

- a) Press and rotate the belt clip at the back of the Elle TENS 2 in either direction to reveal the battery compartment.
- b) Release the battery compartment cover and insert 2 x AA batteries ensuring the positive (+) and negative (-) terminals are correctly positioned as marked in the battery compartment. Replace the cover.

- Take two of the purple leadwires and insert each of the black jack plugs into the sockets at the top of the Elle TENS 2 unit.
- Now insert the pins (at the other end of the leadwires) into the sockets attached to the sticky electrodes. Do not remove the electrodes from the backing at this stage.
- NB. Each leadwire has a red and black pin. It makes no difference which electrode they are attached to.
- e) If you wish, attach the easyrelease neck cord which enables you to hang the Elle TENS 2 from your neck.

Using the Neck Cord for mobility during labour

Thread the loop of the neck cord through the space at the top of the unit. Slip the other end of the neck cord through the loop and pull into position.

Useful Diagrams

